


Interpretation of Results

NOTE: The test results should be read and interpreted at 10 minutes after the sample application and the reading and interpretation of the results should not exceed 15 minutes. The test results should not be interpreted using any instruments.

Positive: two distinct colored lines appear.

One red-colored line next to “C” and one blue-colored line next to “T” indicates COVID-19 positive result.

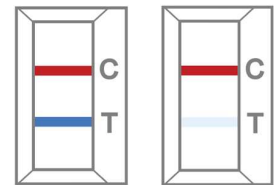

NOTE: The color intensity in the test region will vary depending on the amount of SARS-CoV-2 nucleocapsid protein antigen present in the sample. Any faint colored line(s) in the test region(s) should be considered as positive.

Negative:

One red-colored line only next to “C” indicates a negative result.



Invalid:

If the red-colored line in the control region “C” is not visible, the result is invalid. Re-run the test one time using the remaining specimen in the extraction vial if an invalid result is obtained during initial testing.

